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## ENGLISH • BULK FILL FLOW LIGHT CURE COMPOSITE

**Product description:** It is a low viscosity composite designed to increase aesthetics and easy polishing in Class V, III and IV cavities in anterior teeth. It can be used in cavities where is occlusal loading due to its high particle resistance.

**Clinical benefits:**

- Symptoms relief (pain and discomfort).
- Prevents future complications.
- Very High compressive strength that increase the durability and resistance of restoration.
- Rebuild and return chewing functionality
- Natural appearance and good aesthetics.
- They avoid the use of metals such as mercury
- Duration between 5 and 7 years
- It does not give off harmful components, it is biocompatible with the body.
- Good resistance to wear, fractures and tension.

**Performance characteristics:**

- Excellent properties for molding and sculpting.
- High Brightness.
- Available in any vita color.
- Good polishing.
- It adheres better to the teeth tan other materials.
- Easy to use because it does not require the use of complicated tools for its application.
- It is suitable for all types of patients.
- Bulk Light Cure Composite can be placed on front teeth and molars
- They can be placed in blocks (5 mm increments for class II and 4 mm for class I).
- After polishing the surface to remove any extrinsic stains, determine the shadesneeded for the restoration before tooth preparation or rubber dam placement. A tooth that has been dried appears lighter than normal. For this reason, a shade selected on a dry tooth will be lighter than a tooth after hydration.
- For better results is recommended to use halogen or LED lamp with a wavelength between 400-500nm that activates the photoinitiators of the resin.
- For Hybrid, Microhybrid and Nanohybrid Light cure composites the exposure takes 20-40 seconds per layer (2mm). The exposure time should be increased up to 40 seconds when the layer is thicker than 2mm or if the resin is darker.
- Due to their higher resin content, fluid composites undergo greater contraction during polymerization.

**Shades:** Translucent in all presentations

**Intended use:** Base and liner for direct restorations of Class I and II. For a small Class I direct restorations in permanent molars, and for direct restorations of Class I and II of temporary teeth. Sealing of pit and fissures. Core buildups.

**Indications:** Designed to be used in deep restoration where is possible to put up to 4mm of bulk composite in one action. High transcluency allows a correct curing and a neutral base for the colocation of the final layer of hybrid composite of the adequate shade.

**Intended user:** Dental professionals only with an academic degree, experience, and training for use dental materials, it must be manipulated by a dental professional only.

**Patient population:** All patients of any age, gender, weight, or ethnicity, with permanent dentition, who, for different reasons, must be make restorations with Medental Bulk Flow Light Cure Composite.

**Precautions:** Some patients can present allergy to the components of the formula, in this case, suspend it use. Keep away from Sunlight. Keep away from rain. Keep away the reachofchildren. Keep away from heat or sparks. Temperature limit between 4°C and 24°C. Fragile. Non-Sterile. No requires refrigeration. Only for professional dental use. Room temperature before use. Not requires cold chain to transport. Do not use after date of expiry.

**Contraindications:** The material must not be used when the application techniques described are not possible. Composite resins are subject to irritation in soft tissues and cells in direct contact when the monomers included in their formulation are in direct contact with these tissues, causing irritation and even pulp death if they are not 100% photocured. It is contraindicated to place composite resin in any presentation on materials containing eugenol since these inhibit polymerization. Layers greater than 5 mm should not be placed to allow correct polymerization.

**Composition:** Crystalline and fumed silicon dioxide, Bis-GMA, Stabilizer and Photo initiators.

**Instructions for use:**

**A. TOOTH PREPARATION:** Isolate tooth with rubber dam before starting. Follow usual procedures for tooth preparation. Exposed dentin must be protected by a Calcium Hydroxide preparation. Glass ionomer or another suitable base can be used to cover the preparation. Remove temporary restorations. Isolate and clean the tooth of any traces of temporary cement.

**B. ETCHING TOOTH SURFACE ENAMEL:** Apply etchant over the surface enamel of tooth using a cotton pellet or brush. Allow to remain for 20-30 seconds. On primary tooth high in fluoride a 90-120 second. If Glass Ionomer base is used, do not etch for more than 15 seconds. Flush area completely for 20 seconds. Do not allow patient to rinse the mouth or permit saliva to touch etched enamel surface. Airs dry completely with an oil/moisture free air source. The surface of tooth should have a dull, grayish appearance. If not, repeat the process allowing etchant to remain on tooth for an additional 20-30 seconds.

**C. PRIMING TOOTH SURFACE:**

Apply a thin uniform coating of Dentin Enamel Bonding immediately onto etched tooth surface. Thin the resin coating on the tooth with a gentle stream of oil/moisture free air. Avoid contact with gingival or other soft tissue. If using porcelain faces or veneers, apply a layer of primer for porcelain over the engraved faces or veneers.

**D. RESTORATION:**

Apply the composite from its application tip either from the syringe or from uni-dose tip. Must be placed a layer thickness of 4mm bulk fill in the prepared cavity and polymerize with visible light curing lamp for 40 seconds. If thicker layers are needed for layers of 4mm and polymerize each layer for 40 seconds until the cavity is full as necessary. Then apply a layer of universal hybrid composite polymerize for 20 seconds. Give the usual finishing.

**E. FINISHING:**

Shape and polish the restoration with finishing discs or suitable polishing tools. We recommended the use of silicon diamond abrasives.

**Conservation:**

- The syringe is designed to be used at room temperature above 24°C or 75°F.
- Shelf life at room temperature is two years for unopened kits and one year for opened kit when syringe and vital caps are replaced immediately after dispensing.
- Unopened kits can be refrigerating in (4°C-40°F) to extend shelf life. If stored in refrigerator, bring to room temperature prior to use.
- Do not store materials in proximity to eugenol containing products.
- Do not expose materials to elevated temperature or intense light.

**Warnings and Precautions:** In case of any change on the original function of the product, stop using it and contact the manufacturer. Read carefully and follow all instructions. Before the use of these products always use rubber dam gloves and protective glasses. The etching solution contains a 37% phosphoric acid solution, which can be harmful to skin and eyes. In case of contact with the eye, wash immediately with abundant water for at least 15 minutes and if the irritation persists consult a doctor. In case of contact with skin wash the area with soap and water. The product should not be ingested or aspirated. If accidentally swallowing induce vomit immediately and get medical attention. In case of inhalation, clean the nose with water and seek medical assistance. Ask for patient to rinse his mouth immediately after treatment. The product is designed to be used at room temperature. If refrigerate, the material may be difficult to extrude. It is recommended to check the polymerization time indicated by the manufacturer, according to the layer thickness, because a non-polymerized resin can cause irritation or allergic reactions. Requires following the dentin preparation procedure to reduce gingival recession.

**Warnings:** If the use of this product on a patient or anyone else causes some anomaly such as:

- It does not fulfill the intended purpose.
- It doesn't return the functionality of the tooth,
- It doesn't have the properties it refers to,
- It has damage to packaging or manufacturing,

- The instructions for use are misunderstood.
- Any secondary reaction or anomaly: Please send detailed report to: Conamco, S.A. de C.V. email: [atencionclientes@conamco-sacv.com.mx](mailto:atencionclientes@conamco-sacv.com.mx)

MEDENTAL and/or CONAMCO S.A. de C.V. warranty the product against any fault of manufacture of the material. MEDENTAL and/or CONAMCO S.A. de C.V. do not take responsibility for the use of the product outside of what is specified in this instruction sheet. Being the only obligation is the replacement of defective product.

The summary of Safety and clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at: <https://ec.europa.eu/tools/eudamed>

**Disposal Considerations:**

- Use New and clean Nitrile Gloves.
- The resin must be cured in a safe container for resin, either by exposing it to the sun or with a light cure lamp.
- Once cured, it can be discarded with solid residues.

Note: Liquid or partially cured resin must be disposed of in accordance with the regulations.

**Features/Claims:** Dental Light Cure Composites are part of Dental Surgery and are composed of different synthetic materials that adhere to the surface of the tooth and give it a color similar to the natural one, so its use results in a practical and cosmetic restoration. The Light Cure Composites can be used for a fractured, cracked or fissured tooth, holes, cover stains, biting and chewing problems, fill cavities caused by cavities, improve the color of the teeth. The specialist must carve the surface of the tooth so that the Resin adheres to it, it is required to etch and apply adhesive later, and the product is applied in the corresponding space and molded as desired. Once the Dental Resin is placed, it hardens with the help of ultraviolet light or blue light. To take good care of the Dental Resin, it is recommended to avoid biting on very hard things so that this material does not splinter. Also moderate the consumption of highly pigmented foods such as coffee and wine, as well as maintain good dental hygiene and use dental protector if the patient has Bruxism.

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